

SIZING YOUR NOVAGUARD

HEAD MEASUREMENT INSTRUCTIONS

STEP 1 - LOCATE THE MID POINT

Run an imaginary line across from behind one ear all away up and across the dogs head to the back of the other ear you will find the mid-point located in the centre of this line. This mid-point is used to establish the HEAD MEASUREMENT.

STEP 2 - ESTABLISH THE HEAD MEASUREMENT

Measure from the mid-point to the tip of the dogs nose

- **DO NOT** follow the contours of the dogs face.
- Measure in a <u>STRAIGHT LINE ONLY</u> (you may need to lift a straight ruler away from the dogs head see below diagrams)

The resulting measurement represents the HEAD MEASUREMENT you will use to size your NOVAGUARD using the table provided.

SIZE MATTERS!

A good fit is critical for the NOVAGUARD to function correctly and to give your dog the best chance of accepting this alternative to the dreaded "CONE OF SHAME"

A SNUG FIT!

FINAL SIZING

Using the tables provided size the NOVAGUARD by HEAD MEASUREMENT AND by CANINE WEIGHT. You may find you have 2 different sizes according to the 2 charts. This is because, as we know, dogs come in all shapes and sizes. Simply pick the smaller size. It is always better to size down when in doubt. When fitting your new NOVAGUARD make sure the neck collar is done up firmly otherwise the dog is less likely to accept the collar. Depending on the temperament of your dog, you may loosen this slightly once the NOVAGUARD has been accepted by your special friend.




NOVAGUARD DIAGRAMS





NOVAGUARD TABLES

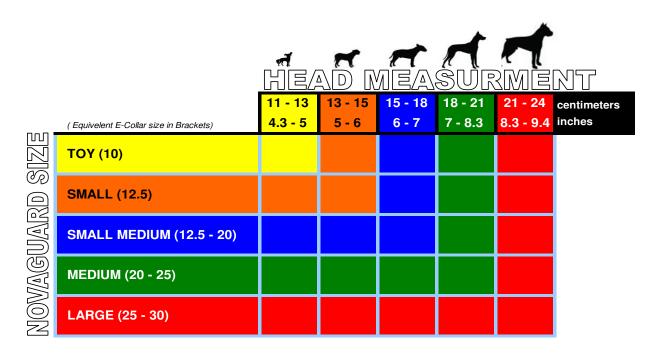

		CANINE WEIGHT					
		2 - 6	5 - 12	10 - 25	23 - 35	33 - 45	kilograms
NOVAGUARD SIZE	(Equivelent E-Collar size in Brackets)	4 - 15	11 - 26	22 - 55	50 - 77	72 - 99	pounds
	ТОҮ (10)						
	SMALL (12.5)						
	SMALL MEDIUM (12.5 - 20)						
	MEDIUM (20 - 25)						
NON	LARGE (25 - 30)						